



Flowflex™

SARS-CoV-2

Antigen Rapid Test (Self-Testing)

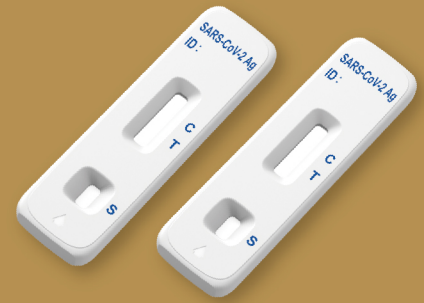

A rapid test for the detection of SARS-CoV-2 antigens in nasal specimens. For self-testing use.

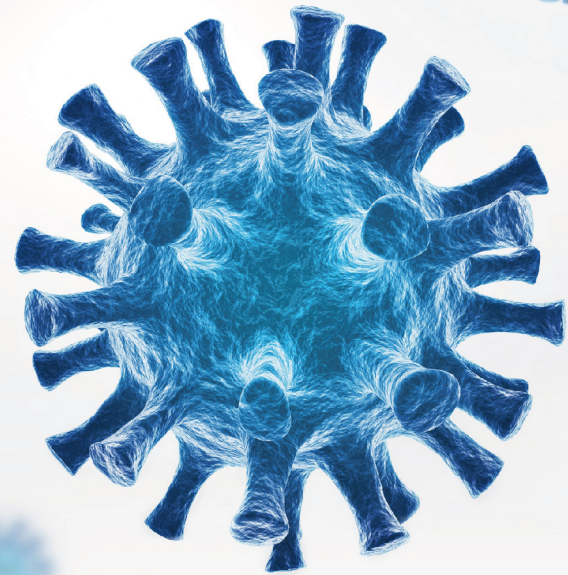

The Global Leader of Rapid Test with **26** Years Experience.



Fast



Accurate



Easy to Use



Reliable

ACON®

Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)

The SARS-CoV-2 Antigen Rapid Test is a lateral flow test for the qualitative detection of the antigen from SARS-CoV-2 in nasal swab specimens directly from individuals suspected of COVID-19 within the first seven days of the onset of symptoms. The test can also test specimens from individuals without symptoms.

- **Specimen:** Nasal swab specimens
- **Test Time:** Results at 15 min
- **Shelf life:** 24 months
- **Storage temperature:** 2-30°C
- **Accuracy:** 98.8%
- **Sensitivity:** 97.1%
- **Specificity:** 99.5%


Test Procedure

1. Test Preparation

A. Open your test kit:





B. You should have:

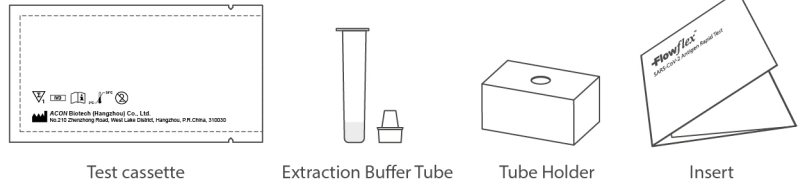


2. Specimen Collection

COLLECTION BY AN ADULT CAREGIVER


SELF COLLECTION

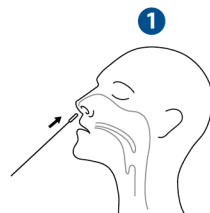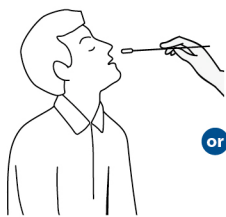


1
Insert the tip of the swab into one nostril. Using gentle rotation, push the swab less than 2.5 cm from the edge of the nostril.

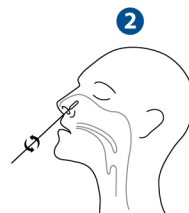

2
Rotate the swab 5 times brushing against the inside of the nostril.

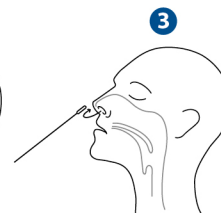


3
Remove the swab and insert it into the other nostril.

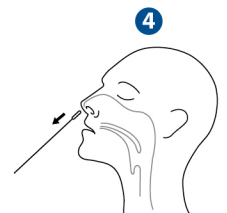


4
Remove swab from the nostril.

3. Sample Preparation

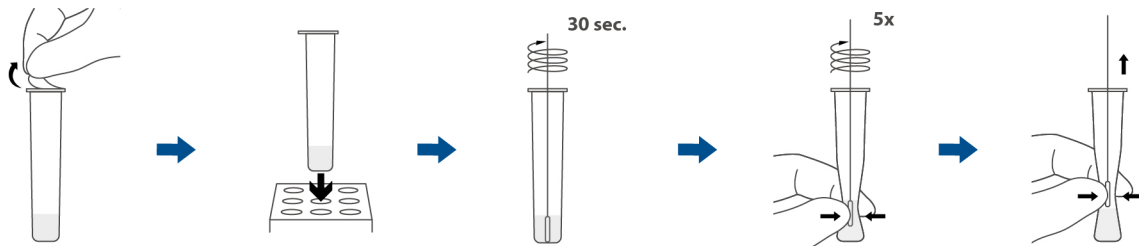

4. Running the Test

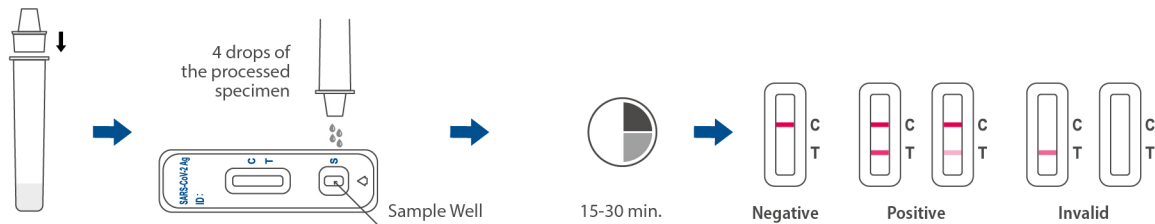

Ordering Information

| Product Name | Catalog No. | Format | Specimen | Package |
|---|-------------|----------|-------------|------------------------|
| Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing) | L031-11855 | Cassette | Nasal swabs | 1 Test/Kit 20 Kits/Box |
| | L031-118A5 | Cassette | Nasal swabs | 5 Tests/Kit |



ACON Biotech (Hangzhou) Co., Ltd.
No.210 Zhenzhong Road, West Lake District, Hangzhou, P.R.China, 310030
Tel: +86-571-8118 9706 / Fax: +86-571-8777 5781 / Email: information@aconlab.com.cn
www.aconbio.com